# InBody Touch 24i

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European Conformity

R 214-240358

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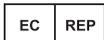
Authorised representative in the European Community

Disposal of old Electrical & Electronic Equipment (Application in the European Union and other European countries with separate collection system.)

This symbol indicates that this product shall not be treated as household waste. Instead, it shall be handed over to the applicable collection point for the recycling of electrical and electronic equipment. By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product. For more detailed information about recycling this product, please refer to local governing ordinances and recycling plans.

FCC ID: F60INBODYTOUCH  
and IC ID: 22967-INBODYTOUCH  
FCC statement

Note : This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

\*This product complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This product may not cause harmful interference, and (2) this product must accept any interference received, including interference that may cause undesired operation.

## Warning

This device contains licence-exempt transmitter(s)/ receiver(s) that comply with Innovation, Science and Economic Development Canada' s licence-exempt RSS(s). Operation is subject to the following two conditions: This device may not cause interference.

This device must accept any interference, including interference that may cause undesired operation of the device.

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Notification for Conformity Assessment of Broadcasting and Communication Equipment  
Registration Number: R-R-KRZ-KIO-AK-R2  
Item Name (Product Name): Kio AK Kiosk (Android Kiosk)  
Basic Model: Kio AK-43-ST(R2) Stand  
Manufacturer/Country of Manufacture: Shenzhen itemp technology / China  
Manufacturer Importer: Krizer co., Ltd.  
Manufactured Date: Separate Indication

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©2025 InBody Co., Ltd. All rights reserved.